



# POINT OF CARE

## Clinitek Status Urinalysis User Instructions

Q-Pulse No	PCM61
Version	1
Site	T

### Quick Reference Guide – Clinitek Status Urinalysis Meter

1. Turn meter on by pressing the on/off  button on the front of the meter. The meter will do a system test which takes about 20 seconds.
2. Press **'Strip Test'**
3. If your badge number is displayed as last operator, press **'Last Operator'**, otherwise press **'Enter New Operator ID'**. Enter your badge number and press **'Enter'**.
4. Press **'New Patient'** square and enter patient name and ID (hospital number). Press **'Enter'**.
5. Take the lid off the urine bottle: get a tissue and urinalysis test strip ready ensuring you have checked the expiry date on the test strips.
6. Dip the test strip into the urine for a count of one, remove from the urine and **IMMEDIATELY** press the **'Start'** button. Gently tap any excess urine from the test strip by tapping its edge against the tissue. This must be done whilst the black squares count down on the screen (10 seconds).
7. Place the test strip in the centre channel of the black test drawer on the front of the meter.
8. The test drawer will move into the meter automatically.
9. After 50 seconds the black test drawer will re-emerge and the results will be displayed on screen. Results will also be printed.
10. Press **'Done'** when the test is complete. The screen will revert to step 2.
11. Repeat from step 2 for any additional urine samples that require testing.
12. When testing is complete remove the top part of the black test drawer, rinse carefully, dry gently and replace.
13. Press and hold the on/off  button to switch the meter off.